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### **Thank you very much and congratulations on your choice**

We are glad you choose a product by hofmann gmbh. We certainly hope that your pain will be relieved by this device. In order to become familiar with the most important functions and features of this device, we ask that you study the following diagrams thoroughly and read the entire instruction manual before use. The medilike® Life Ems is battery operated and has two channels. This device creates electrical impulses, that can be adjusted in intensity, frequency, and duration per second by control switches. The switches are user friendly and the slide cover prevents changes to the settings by accident. We recommend you ask your physician before using the medilike® Life Ems.

All products from hofmann gmbh are subject to constant quality control. However, should you find any fault with the product, should you have any suggestions, or should you simply want to tell us your opinion of this product, feel free to contact us. We are always glad to help.

Yours,

date of edition: 01 April, 2003

## **Introduction to the use of the medilike® Life Ems device**

Electrical muscle stimulation is an internationally accepted and probed treatment method for muscle injuries. It works by sending electronic impulses to the muscle to be treated, which effects passive movement of the muscle.

This is a product developed from the square pulse wave form invented by John Faraday in 1831. By use of square wave pattern it is possible to directly affect the motor sensory neurons of the muscle. EMS has low frequency, which together with the square wave pattern makes direct stimulation of the according muscle groups. It is often applied in hospitals and physical therapy clinics for treatment of muscle injuries and rebuilding of paralysed muscles in order to prevent atrophy and to improve muscle tonus and blood circulation.

## **How does the medilike® Life Ems work?**

EMS is applied to improve blood circulation. It loosens tight and cramped strands of muscle, stimulates muscle growth and reduces stiffness within the muscle connection. The EMS devices send comfortable impulses trough the skin, that stimulate the nerve within the treatment area. When the muscle receives the signal it contracts just as if it receives a signal sent from the brain. While the signal increases in strength, the muscle moves as during physical exercise. When the impulse stops the muscle relaxes and the circle starts anew (stimulation, contraction, relaxation).

## Safety Instructions

Please read the instruction manual before using your medilike® Life Ems for the first time.

- Use your EMS device only according to the instruction manual and to its purpose.
- Do not attach accessories not recommended by hofmann gmbh.
- Patients with electronic (f. ex. pace maker) or metallic implants may not apply EMS without consulting their physician.
- In case the application of EMS is uncomfortable or ineffective, the stimulation must be stopped until examination by a physician.
- Do not use the device while operating machines or driving a vehicle.
- Please always turn off your device before application or removal of the electrodes.
- EMS units do not have a AP/APG-safety. They may not be operated around explosive or combustible material.
- Patients with a heart condition are advised to only use EMS with special care. Consult your physician about side effects before applying the device.
- EMS may not be used during pregnancy.
- The use according to the purpose of EMS impulses is nerve stimulation. If the device is used on unsuitable body parts danger to the patient can result. Never place the electrodes in the laryngeal/pharyngeal area of the throat. Muscle contractions in this area can cause asphyxiation.
- In some cases there have been reports of skin irritations caused by the electrodes. In this case please stop stimulation and remove the electrodes. Wait until clearance of the irritation's cause before continuing treatment.
- Do not apply electrodes to your eyes, in your mouth, or in other body holes. Only use electrodes on intact skin. Never apply electrodes on abrasions or irritated skin.
- Keep away from children.
- Do not apply electrodes in a fashion that electricity flows through the brain.
- The device may not be used to treat pain of unknown cause.

**Specific information valid in the Federal Republic of Germany:**

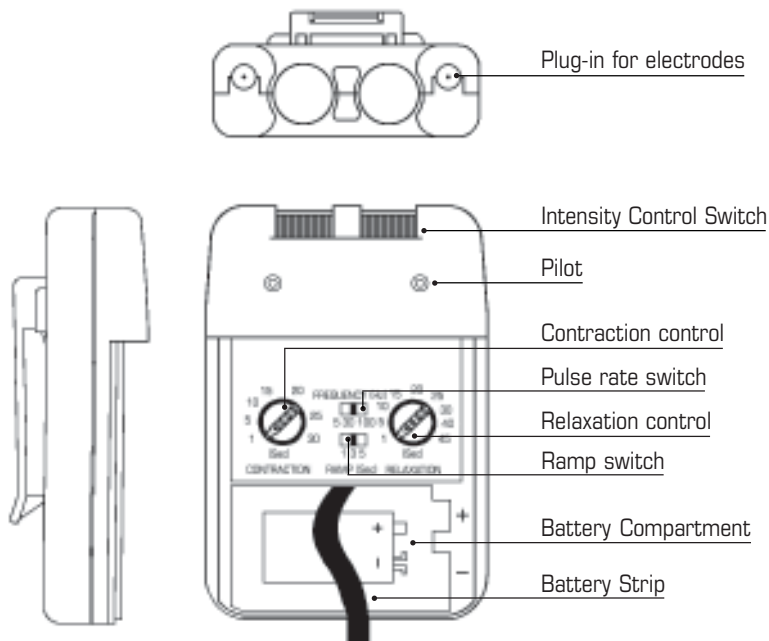
- The device is in accordance to the Law for Medical Products (MPG).

The device is subject to the Law for Medical Product Operators (MPBetreibV.)

Operator and user are obliged to observe the Operator's Law in complete contents.

For all other countries within the European Community the national Laws for operating a medical product apply.

## The medilike® Life Ems

## **Use of the setting elements**

You should be familiar with the different functions of your medilike® Life Ems device through reading the previous chapter. This chapter introduces the individual switches and explains their purpose and function.

### **Intensity**

Each patient reacts individually to the different intensity levels according to differences in tissue resistance, enervation, thickness of skin, a.s.f. The instructions regarding intensity are therefore restricted to the following settings:

#### **Sensory perception**

Intensity is increased to a level where the patient can feel the stimulation, but without ensuing muscle contraction.

#### **Light muscle contraction**

Intensity is increased to a level where hardly visible muscle contractions ensue not strong enough to move a joint. At application of low-pulse-setting twitching can be noted. At a higher pulse rate simply the muscle tonus is increased.

#### **Strong muscle contraction**

This method of muscle stimulation can be especially applied for spastic muscle groups to release the spasm. Choose a fast pulse rate as well as wide pulse duration setting and regulate the intensity to visible contraction that is still within patient's pain tolerance. Twenty to thirty minutes of treatment with this method usually ends the muscle spasm. In case of stimulation being uncomfortable it should be stopped or intensity be reduced.

### **Contraction/relaxation**

The EMS contraction and relaxation times are variable. Stimulation ensues during set time and is followed by a relaxation phase according to your individual setting. This cycle repeats –stimulation, contraction, relaxation.

### **Pulse rate setting**

By means of this switch you may determine how many electrical impulses are sent through your skin within one second. By variation of the switch position the amount of running impulses per second (Hz) can be set for both channels. There are 3 setting options: 5, 30 and 100 Hz.

### **Ramp switch**

In order to prevent an uncomfortable treatment start there is the possibility to adjust the course of contractions ascending. This effects the impulse to build softly and slowly. Intensity of the electrical current reaches the highest value within the chosen time of intensity increase. In case total treatment time is shorter than the time set for intensity increase, the highest value will not be reached.

### **Time duration**

The medilike® Life Ems device can be used over long periods of time, usually 20 to 30 minutes per session. Ask your physician for the optimum treatment duration.

### **Attaching the electrode lead wires to the device**

The wires provided with the medilike® Life Ems insert into the jack sockets located on top of the device. Holding the insulated portion of the connector, push the plug end of the wire into one of the jacks (see drawing); one or two sets of wires may be used. After connecting the wires to the stimulator, attach each wire to an electrode (positive and negative pole). Use care when plugging in or unplugging the wires. Jerking the wire instead of holding the insulated connector body may cause wire breakage.

### **Caution**

Do not insert the plug of the lead wire into AC power supply socket.

### **Lead wire maintenance**

It is important that you take good care of the lead wires. Failure to do so will result in reduction of the device's overall effectiveness. Clean the wires by wiping with a damp cloth. Coating them lightly with talcum powder will reduce tangling and prolong life.

### **Electrode options**

Your physician should decide which tape of electrode is best suitable for your condition. Follow application procedures outlined in the manual of the electrodes to maintain stimulation and prevent skin irritation. Only CE approved electrodes may be used.

## **Placement of electrodes**

One of the most important factors for achievement of successful pain therapy is the placement of the electrodes. Willingness of the physician, as well as the patient, to try different electrode placement options in order to find out the most effective treatment method is also eminent.

Be prepared to invest a little time and energy in experimenting with electrode placement. It will increase your chances for successful treatment. Every patient react differently to electrical stimulation and individual needs may differ from the conventional settings. If your first results are not yet positive, you ought to try different settings. In the following section the variables of electrode placement are outlined. Do experiment with the different strategies or ask your physician which placement technique might be most effective for you.

## **Contiguous placement**

This is the most common placement technique. Electrodes are applied close to or around the localized area of pain so that the electricity flows through it. This is achieved by placing an electrode on each side of the inflicted area. This works well if the pain is localized on an extremity or deep within the tissue. Pad placement on the front and back of the affected limb allows the current to flow completely through the limb and thus through the pain area.

## **Dermatomes, Myotomes, and Sclerotomies**

These are regions of the body that are enervated by a spinal nerve. The technique of electrode placement involves stimulation across the enervated area, achieved by placing one electrode on the pain site and another at the point where the nerve root joins the spinal cord.

### **Motor, Trigger and Acupuncture Points**

These are points of high tissue conductivity that can individually vary in location. Theoretically the electrode placement is the same as described above. The easiest technique involves placing one electrode directly on the spot of pain and completing the circuit by placing another in the area on the affected side. This second electrode site can be within a nerve zone, or can be the master point located between thumb and forefinger on the dorsal web area of the hand between the two metacarpal bones.

### **Multiple placement strategies**

As this EMS device has two different channels, you may take advantage of alternative, concurrent placement strategies. For example, it is possible to use two different placement strategies at the same time. One channel can be used to directly stimulate the pain site in a contiguous manner, while the other channel can be utilized for point therapy.

## Advice on skin care

To avoid skin irritation, especially if you have sensitive skin, follow these suggestions:

- Wash the area of skin where you will place the electrodes, using a mild soap and water, before applying the electrodes and after taking them off, being sure to rinse the soap thoroughly and to dry the skin well.
- Excess hair may be clipped with scissors; however do not shave the area to be stimulated.
- You may chose to use a skin treatment or preparation. This should be recommended by your physician. Apply, let dry, and then apply the electrodes as directed.
- Many skin problems arise from the „pulling stress“ caused by tight application of the adhesive pads. In order to prevent this, apply electrodes from centre to edge and avoid stretching over your skin.
- To minimize „pulling stress“ tape the extra length of the lead wires to the skin in a loop, thus preventing possible tugging on the electrodes.
- When removing the electrodes always pull in the direction of hair growth.
- It may be helpful to rub skin lotion on the area of placement between applications.
- Never apply electrodes to already irritated or broken skin.

**How to apply self-adhesive electrodes**

- Clean skin at the prescribed area thoroughly with soap and water and dry well prior to application of electrodes.
- Insert the lead wire into the pin connector on the prewired electrodes.
- Remove the electrodes from the protective liner and apply firmly to the treatment area.

**How to remove self-adhesive electrodes**

- Lift electrodes at the edge and peel off; do not pull on the lead wires as this may damage the electrodes.
- Place the electrodes on the liner and remove the lead wire by twisting and pulling at the same time.

**Care and storage of self-adhesive electrodes**

- Between uses store the electrodes in the resalable bag in a cool, dry place.
- Improve life span for repeated use by spreading a few drops of water over the adhesive and turn the surface up to air dry. Note, however, that overly saturating with water will reduce the adhesive properties.

**Caution**

- Do not apply to broken skin.
- The electrodes should be discarded when no longer adhering
- The electrodes are intended for single patient use only.
- If irritation occurs, discontinue use and consult your physician.
- Read all instructions for use of self-adhesive electrodes before application.

## **Adjusting the settings**

### **Panel cover**

A cover conceals the controls for pulse width, pulse rate, mode selector and modulation selector. Your medical professional may wish to set these controls for you and request that you leave the cover in place. Press the top side of the cover and pull down in order to open the cover.

### **Display LED**

The EMS device's LED illuminates each time the device releases an electrical impulse. Note, the LED illuminates for frequencies up to 30 Hz. At higher frequencies the LED will appear illuminated continuously.

### **On/Off switch and intensity control**

If both intensity control knobs are in off position the device is switched off. Turning the control knobs clockwise turns on the device. Once the device is turned on the LED should begin to display the electrical impulses. Electrical pulse increases in intensity as the controls are turned clockwise. To reduce the strength of electrical pulses or turn off the device, turn the control knobs counter clockwise to the required setting or to off position.

### **Contraction control**

Time duration of stimulation is set by the contraction control. By turning the switch the time of contraction can be chosen. Setable time ranges from 1 to 30 seconds.

### **Relaxation control**

Relaxation duration is set with this switch. Stimulation is interrupted at the chosen time by a phase of relaxation to continue in a cycle. Relaxation time for both channels can be set with this switch. Time duration is variable from 1 to 45 seconds.

**Pulse rate switch**

This switch regulates how many electrical impulses per second are sent through the skin. By means of this switch the amount of impulses per second (Hz) can be determined for both channels. This switch has three steps: 5, 30 and 100 Hz.

**Ramp switch**

This switch sets the time for intensity at the current output. At the intensity increase setting each contraction is conducted increasingly, so that the impulses ensue softly and slowly. There are three settings possible: 1, 3 and 5 seconds.

**Check/Replace the battery**

In order to insure the functional safety of your EMS device, over time a battery change is necessary.

- Make sure both intensity controls are set to off position.
- Slide open the battery compartment cover.
- Remove the battery from the compartment.
- Insert the new battery in the compartment. Note the polarity indicated on the battery and in the compartment.
- Replace the battery compartment cover and press to close.

**Setting recommendation for your medilike® Life Ems**

Please set your medilike® Life Ems as follows:

- Contraction control (Contraction) set to about 5 seconds
- Relaxation control (Relaxation) set to about 20 seconds
- Pulse rate control (Frequency) set to 100 Hz
- Ramp switch (Ramp) set to 1 second

## **Battery information**

If you use rechargeable batteries, please be sure to follow these instructions.

### **Rechargeable batteries**

Prior to the use of a new unit the rechargeable battery should be charged according to the battery manufacturers instructions. Before using the battery charger read all instructions and cautionary markings about the battery as well as this manual carefully.

After storage of 60 days and longer, the batteries may loose their charge. Therefore, in case of long periods of storage, prior to use, the batteries ought to be recharged.

### **Battery charging**

- Plug your battery charger into any working 220/230V electrical outlet. The use of any attachment not supplied with the charger may result in risk of fire, electric shock, and injury.
- Follow the battery manufacturer's instructions on charging time.
- After the battery manufacturer' recommended charging time has elapsed, unplug the charger and remove the battery.
- Batteries should always be stored in a fully charged state.

To ensure optimum battery performance, follow these guidelines:

- Although charging the batteries for up to 24 hours will not damage them, repeated overcharging may decrease battery lifetime.
- Do not short the terminals of the battery. This will cause the battery to get hot and can cause permanent damage. Avoid storing the batteries in your pocket or purse as the terminals may accidentally come into contact with coins, keys, or other metal objects.

- **Caution**  
Do not recharge other batteries but the nickel-cadmium batteries. Other types could leak or explode. Do not expose rechargeable batteries to heat sources or fire, as they may explode!

### **Caring for your medilike® Life Ems, transportation and storage**

- The device may be cleaned with rubbing alcohol. Advice: When working with flammable liquids, do not smoke, or work around an open flame (for example lit candles)
- Stains and spots can be removed with a mild cleaning agent.
- Do not immerse the unit in liquids or expose it to large amounts of water.
- Always return the device to the carrying case in order to ensure that the unit is well protected for transportation.
- If the device is not used for a longer period of time, remove the batteries from their compartment, because acid may leak and damage the device. For storage place the device and the accessories in the carrying case and store at a cool and dry place.

## **Safety technical controls, STC**

According to the Law for Medical Products Operators (MPBetreibV) § 6 (Safety-Technical Controls) the Operator is obliged to conduct regular safety-technical controls. hofmann gmbh prescribes according to §6, MPBetreibV, these controls in a 24-months turn.

The safety-technical controls are to be conducted by personnel only, that is based on their education, knowledge and practical experience proficient, that is independent, and that utilizes suitable measuring and proving devices.

The prescriptions for safety technical control is to be requested from the quality manager of hofmann gmbh.

### **Prior to application of the medilike® Life Ems the following checks are to be conducted:**

1. Check the device for external damage:
  - deformation of body
  - damaged or defective output sockets.
2. Check the device for defective operating elements:
  - legibility of inscriptions and labels
  - make sure the inscriptions and labels are not distorted.
3. Check the LED
  - LED must be illuminated when unit is switched on.
4. Check usability of accessories:
  - Electrode cable undamaged
  - Electrodes undamaged (no splits, no breaks)

## Malfunctions

Should any malfunctions occur while using the EMS, please check:

- Are switches and controls set for the appropriate treatment modus? Adjust accordingly and correctly.
- Are the wires connected correctly to the device? They should be inserted completely into the sockets.
- Is the impulse LED illuminated? If necessary, insert a new battery.
- Are the wires damaged? If any damage to the wires is noted, they need to be replaced.
- Are two electrodes connected to the pair of wires?

If there are any other problems with the unit, please return it to the distributor. Do not try to repair a defective device yourself.

Please consult your distributor for problems with your device or the accessories.

## Advice on recycling

The packaging ought to be disposed of at the garbage recycling container, the metal parts of the device at the used metal recycling station. Plastic pieces, electronic parts, and conducting boards are to be disposed of as electronic waste. Batteries go to the special garbage collection. Please render them to your Recycling plant and do not throw them into your regular household waste. For further information about recycling and disposal of special waste turn to your local recycling plant.

**Your medilike® Life Ems includes the following contents:**

- 1 x medilike® Life Ems unit
- 2 x lead wires
- 4 x self-adhesive electrodes, size 40 x 40 mm
- 1 x battery, 9 V
- 1 x instruction manual
- 1 x carrying case

**Caution**

The above mentioned contents are standard equipment, that can vary within distributors.

**Accessories for your medilike® Life Ems (partly not included in the package):**

- self-adhesive electrodes, 2mm plug, 40 x 40 mm, article No. 50410
- self-adhesive electrodes, 2 mm plug, 40 x 80 mm, article No. 50100
- self-adhesive electrodes, 2 mm plug, size 32 mm round, article No. 50050
- permanent electrodes for gel pads, article No. 50430
- gel pads, size 40 x 40 mm, article No. 50470
- cables for permanent electrodes, article No. 50420
- cables for self-adhesive electrodes, article No. 50400
- rechargeable 9 V accumulator, article No. 50450
- charger for 9 V accumulator, article No. 50460

If you would like to make your device compatible with the permanent electrodes, you need to supply yourself with the fitting cables and the electrode gel, respectively gel pads.

## Technical specifications

Channels:	Dual, isolated between channels.
Pulse Amplitude:	Adjustable to 0-80 mA peak into 500 Ohm load each channel, constant current.
Pulse rate:	5Hz, 30Hz, 100Hz (adjustable with the pulse rate control)
Pulse duration:	250 $\mu$ s (fixed)
Contraction time:	1 sec. - 30 sec. (adjustable)
Relaxation time:	1 sec. - 45 sec. (adjustable)
Ramp time:	1, 3 und 5 minutes (adjustable)
Wave Form:	asymmetrical bi-phasic square pulse
Voltage:	0-100 V (open circuit)
Power Source:	9 V battery
Battery Life:	approximately 60 hours at regular settings
Operating Conditions:	0°C to +50° C at 20 to 75% relative humidity, however with no condensation, pressure 700 hPa to 1060 hPa.
Storage Conditions:	0 ° C to +50° C at 20 to 75% relative humidity, however with no condensation, pressure 700 hPa to 1060 hPa.
Dimensions:	95 mm (height) x 65 mm (width) x 23,5 mm (length)
Weight:	115 g (inclusive of battery)
Safety for electrical shock, type BF.	
All electrical specifications valid at $\pm$ 20 % 500 Ohm load.	

## Explanation of signs on the CE label

Observe the instruction manual!

Application component of type BF. Application components are not suitable to be used for direct application at the heart.

## **Warranty**

There is a 2 year guarantee for our products, applicable from date of purchase.

Within the prescribed warranty time we will remove all defects of the device, that are based on faulty material or manufacture. Removal of defect occurs at our choice by either repair or replacement of parts or the entire device.

Exempt from warranty are: Defects caused by inappropriate use (f. ex. connection to inappropriate electrical current or voltage, connection to inappropriate sources of electricity, breakage by fall and such). Normal wear and tear that only minimally affects the operational utility of the unit is not subject to the warranty.

Warranty subsides if unauthorized intervention or repair is undertaken, or if non-original parts are used.

Warranty is effective only, if date of purchase is confirmed on the warranty card by stamp and signature of the dealership.

In case of warranty claims, please return your device completely with the warranty card to one of our authorized dealerships or send both to the nearest customer service.

No other claims are valid under the provisions of this warranty, unless our liability is based on legal statute. Claims against the dealer based on the provision of the purchase agreement do not affect this warranty.

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